

Thank You!

Peak Folding Wheelchairs are designed with the user in mind so please take the time to go through this manual before using your new wheelchair.


Keep this booklet handy and use it when required.

The Peak Folding Wheelchair is warrantied to be free of defects in material and workmanship for twelve months from the date of purchase. This device was built to exacting standards and has been carefully inspected prior to packing and shipping. This warranty is an expression of our confidence in the materials, our workmanship and an assurance of the same to our customers.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the unit. This warranty does not cover device failure due to owner's misuse or negligence or normal wear and tear. This warranty does not extend to nondurable components such as wheels and brakes which are subject to normal wear and need periodic replacement.

Peak Care Equipment 149 Princes Highway Unanderra NSW 2526 Tel: +612 4272 2688

www.peak-care.com.au

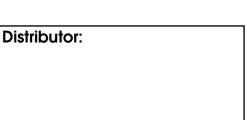



Peak Neptune Self-Propelled Wheelchair

Product Code: 9346376060516 9346376060721 (Red)



User Manual





Instructions:

 To open: push down on seat rail. Do not put your fingers between seat rail and sideboard



 To fold Wheelchair you should lift up the footplates and then remove the legrests



 Adjust footplate with the footrest bolt



 Use the handrims to move the wheelchair



 Press the back tube end of chair down to lever front of chair



 Do not stand on fooplate or seat

2



 Hold wheelchair tightly on slope



 To remove armrest press the bottom lock and lift the armrest



 Lock wheel by pushing brake handle forward



 To remove legrest, push lock-plate out of the legrest hanger





For Your Safety



- 1. Engage wheel locks against both rear wheels before sitting down or getting up from the chair
- 2. Speed limit while going down a slop shall not exceed $8\mbox{km/h}$
- 3. Do not use the wheelchair on bumpy surfaces
- 4. Do not use the hand lock to stop the chair when it is moving

3