

ERGOADJUST DELUX

MODERN LIVING

Avante
INSPIRED LIFESTYLE

- Bluetooth Application *available separately*
(Control the bed with your phone!)



The ErgoAdjust Deluxe is Avante's new advanced adjustable bed that embodies function and comfort. The bed is equipped with a full package of functionalities for mobility including Hi-Lo lift and pre-set customised positions. The ErgoAdjust Deluxe is also designed to maximise user experience, featuring vibration massage, head tilt, USB charging ports, under-bed lighting all controlled effortlessly with a wireless handset. The ErgoAdjust Deluxe is the optimal choice for anyone seeking independent mobility and a better sleeping experience.

- Hi-Lo Function
- Vibration Massage
- Head Tilt Position
- USB Charging Ports
- Low Point: 380mm
- High Point: 670mm
- Under-bed Lighting
- Wireless Hand Control
- Customisable Pre-Set Positions

SIZES AVAILABLE

HALF QUEEN
LONG SINGLE
KING SINGLE
LONG DOUBLE

* 10 Year Guarantee applies to frame, structure and electric motors only. All other electronic components and upholstery are covered for 2 years.

ERGOADJUST DELUX ADJUSTABLE BED MANUAL




INSPIRED LIFESTYLE

TABLE OF CONTENTS

Safety	1
Introduction	3
Care	4
Features	5
Unpacking Your Base	6
Attaching Castors	7
Attaching Mattress Bars	8
Hand Control Overview.....	9
Troubleshooting	12
Product Guarantee	14
Notes	16

SAFETY

ATTENTION: Important Safety Disclaimers
Read all instructions before using
your adjustable base. Save these instructions.

WARNING

Always unplug the base from electrical outlet before servicing any part of the base. Unplug the base before cleaning. To safely disconnect, ensure base is in a flat position with all motors off.

TO REDUCE THE RISK OF SHOCK, FIRE OR INJURY: For optimal safety and operation, plug the bed base into a surge protector (not included). Ensure to always unplug the base from the electrical outlet before servicing or cleaning any part of the base. Keep the power cord away from heated surfaces. Discontinue use of the bed base and contact a qualified service centre if it has a damaged cord or plug, if it is not working properly, or it has been dropped into water. Only use this bed base for its intended use as described in this manual. Do not use accessories/attachments that are not recommended by the manufacturer. Improper connection of the equipment can result in the risk of electrical shock, electrical fire or fault if the plug does not fit your outlet. Contact a qualified electrician to install a suitable outlet.

PETS AND CHILDREN: Dispose of all packing materials immediately after opening as it may pose a risk to small children and pets. To avoid injury, it is not advised to allow children and small pets to play under the bed. Children should not operate the bed base without adult supervision.

SAFETY

- To reduce the risk of falls on height adjustable bases, the base should always be in its lowest position for sleeping and resting. The highest position should only be used for carer's access to the user, ease of entry and exit of bed for the user and ease of making the bed.
- A minimum of 2 able and fit adults should be present when lifting or moving the bed. It is recommended to use trolleys or other lifting/moving devices where possible.
- Careless use of naked flames, cigarettes or other heat sources could set fire to this product.
- Do not operate whilst any person is under the base whilst raising/lowering or adjusting the bed.
- Care should be taken that no body parts are in or near the adjustable frame while operating.
- Ensure the cabling of the bed is free from the mechanism.
- Don't run over electrical cables when moving the bed.
- Do not manipulate or conduct electrical maintenance on the motors or any of the beds electrical components or cables. Any electrical work must be facilitated by a qualified tradesperson.

WARNING

- Stay clear of moving parts while in motion.
- Before adjusting, make sure children and pets are clear of moving parts and are not under bed base.
- Do not allow children to operate bed base without adult supervision.

INTRODUCTION

Congratulations on the purchase of your new **AVANTE ERGOADJUST DELUX BED**

The ErgoAdjust Delux is Avante's new advanced adjustable bed that embodies function and comfort. The bed is equipped with a full package of functionalities for mobility including Hi-Lo lift and pre-set customised positions. The ErgoAdjust Delux is also designed to maximise user experience, featuring vibration massage, head tilt, USB charging ports, under-bed lighting all controlled effortlessly with a wireless handset. The ErgoAdjust Delux is the optimal choice for anyone seeking independent mobility and a better sleeping experience.



Key Features:

- Hi-Lo Lift Function
- Vibration Massage
- Head Tilt Function
- Under-Bed Lighting
- USB Charging Ports
- Wireless Hand Control
- Customisable Pre-set Positions
- 200kg weight rated

CARE

GENERAL CARE INSTRUCTIONS

- Legs and castors should be checked for tightness regularly.
- When transporting - do not over tighten when restraining.
- Do not exceed the maximum weight limit for this bed (200kg total).
- Always lock castors other than when moving or relocating the bed.
- Do not jump on or abuse the bed.
- Do not constantly push the buttons on the controller when not for intended use.
- Use the controller only for adjusting the bed frame for intended use - not for entertainment.
- Lubricate moving parts using a non-corrosive lubricant every 6 months or as required. Avante recommends the use of INOX.
- Beds should be positioned with clearance from walls and other objects so as not to cause damage.
- Do not rock or tilt the frame.

FEATURES

PRODUCT FEATURES

- Stylish modern living design
- Adjustable head and foot parts
- Head tilt function supporting neck muscles
- Massage function
- Wireless multifunction remote control
- USB charging ports
- Under-bed lighting



Head Tilt Function

FRAME DESIGN FEATURES

- Whisper quiet, low voltage, DC lift system provides smooth and efficient operation.
- Long lasting steel support.
- Superior steel foundation.
- Powder coated steel.
- Designed to lift up to 200kg total weight
- Gravity fall motors.
- Can suit any adjustable bed mattress.
- Wall aligned technology.




Raised Position with Head and Foot Up

UNPACKING YOUR BASE

1. Open the carton box & remove the protective plastic.



2. Locate box of parts, remove zip ties.

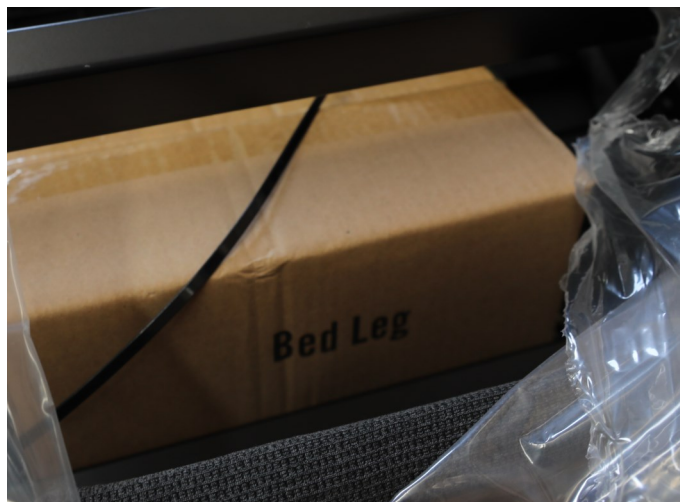

3. Locate hand controller, remove zip tie and protective plastic.



ASSEMBLING (CASTORS)

1. Turn your ErgoAdjust Delux base on its side.
2. Locate the four castors from the box of parts; before screwing in, ensure all castors are in a locked position.
3. Screw in the castors into each hole at the corners of the bed as shown below.



ASSEMBLING (MATTRESS BARS)




















1. Turn the base back to a flat position, with castors now attached.
2. Locate the mattress bars; attach each mattress bar as shown below.



HAND CONTROL OVERVIEW

The remote should look like the one to the right. The remote keys have backlight for assisting operation at night.

Please note—to transfer between saved positions, return the bed to FLAT position FIRST, before selecting next position.

- | | | | |
|---|-------------------------|---|------------------|
|  | Zero Gravity Position |  | Pairing Button |
|  | Anti-Snore Position |  | Raise Bed Height |
|  | Memory Position |  | Lower Bed Height |
|  | Head Up |  | Head Down |
|  | Foot Up |  | Foot Down |
|  | Head Tilt Up |  | Head Tilt Down |
|  | Flat Position |  | Head Massage |
|  | Massage Time Switch |  | Foot Massage |
|  | Turn Massage On and Off |  | Underbed Light |
|  | Massage Type Switch | | |



ERGOADJUST DELUX
HAND CONTROLLER

RE-PROGRAMMING HAND CONTROL POSITIONS


STEPS OF RE-PROGRAMMING PRESET POSITION

1. Adjust the bed into desired position.
2. Choose the pre-set position that you would like to change (Zero Gravity or Anti Snore).
3. Press and hold the chosen pre-set button and the flat position button simultaneously for 5 seconds, until the backlight begins to flash. The current position of the bed is now saved as a custom position under the pre-set button.

SET UP MEMORY POSITION

1. Adjust the bed into desired position.
2. Press and hold the “Memory” button and the flat position button simultaneously for 5 seconds, until the backlight begins to flash. The current position of the bed is now saved as a memory position

PRESS AND HOLD
TO CHANGE PRE-SET POSITIONS

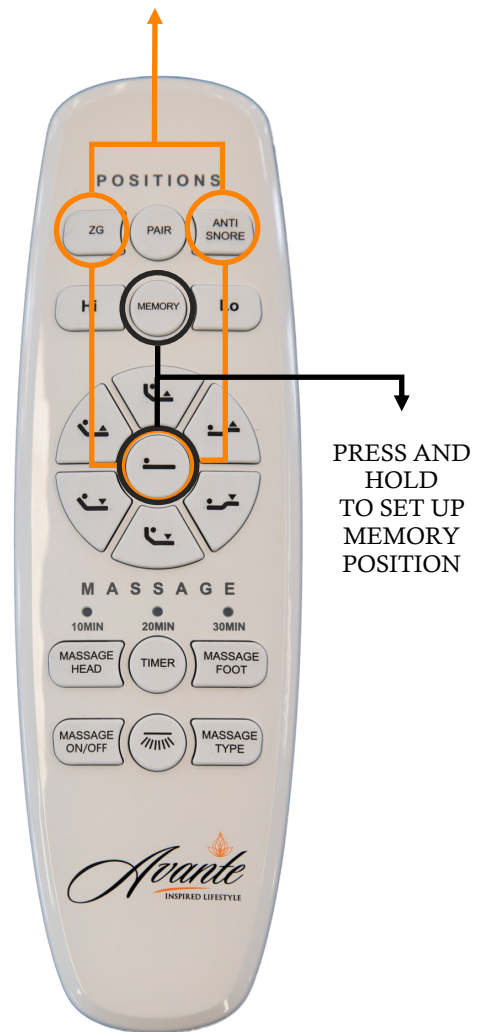

ERGOADJUST DELUX
HAND CONTROLLER

SYNCING BASES

To Sync ONE base ONLY

1. Insert 2 AAA Batteries into the hand controller.
2. Connect power to the transformer.
3. Press the pair button on the control box twice quickly (the blue pairing light will turn on).
4. Press and hold the pair button on remote control for 3 seconds. The back-light will start flashing. The blue pairing LED light on the control box will turn green to indicate that pairing is complete. If not successful, repeat the above steps.

PRESS AND HOLD



ERGOADJUST DELUX
HAND CONTROLLER

To Sync TWO bases to ONE remote

Requires Link Cable

1. Insert link cable end into each control box on the separate bases you wish to sync.
2. Once completed, either remote will then operate both bases.

TROUBLESHOOTING

Below is a list of possible problems & their solutions should the adjustable base fail to operate properly

PROBLEM	SOLUTION
The base does not respond when the remote is used. Remote LED illuminates and appears to be normal.	Perform any or all of the following: <ul style="list-style-type: none">• Check that the power cord is plugged into a working & grounded electrical outlet.• Check that the light on the transformer is on.• Check the home's circuit breaker & fuses.• Unplug the power cord from the outlet. After one minute, plug it back in. Press any button on the remote.
The remote does not work or does not light up.	<ul style="list-style-type: none">• Check the remote LED light is lighting up when pressed. If not, please replace 2 x 1.5V AAA batteries.
Base stutters while in motion.	<ul style="list-style-type: none">• Clear any clutter or obstructions under & around the adjustable base. <i>Radio frequency interference from other remotes or electronic devises may affect normal operation.</i>• Ensure the buttons on the remote are being pressed firmly down when operating the Head & Foot functions.

TROUBLESHOOTING

PROBLEM

SOLUTION

Massage motor/s seem excessively loud during operation.

- Hard surface flooring may cause added vibration. Place carpet pieces or rubber furniture cups under each leg or caster of the bed.
- Check that the bed is not in direct contact with a wall, or other object that may cause vibration or noise.
- If adjustable base is installed on top of a bed frame, check that the massage motors are not causing the bed frame (or components) to vibrate or rattle.
- If a headboard is utilised, please check that the attachment hardware is tightened firmly.

USB and/or under-bed lighting.

- Check to see if the cable has been pulled out from control box. If so, plug back in.
- For USB, try using alternative devices and/or cables. **Please note:** Generic cables may not work and you may be required to utilise genuine cables manufactured by your devices company. **The USB port is a SLOW charging port and will not support Fast Charge.**

*If you have followed the instructions outlined in this manual, and your problem is persisting, please contact the **retailer** you originally purchased your bed from or **Avante Australia** for further assistance.*

PRODUCT GUARANTEE

Congratulations on your purchase. Avante Australia is an International Brand that uses technology, production facilities and teams with over 20 years' experience.

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.

Avante Australia proudly provides a full guarantee (10 years—applies to frame, structure & electric motors only. All other electronic components & upholstery are covered for 2 years). Please refer to www.avante.com.au for details against faulty workmanship and/or faulty materials. During the guarantee period, if we are satisfied there is a genuine fault, we will repair or replace the product if its an option.

THE GUARANTEE DOES NOT EXTEND TO THE FOLLOWING:

- Fabric (including stains, soiling, burns and normal wear).
- Items upon inspection that are deemed unsanitary.
- Normal wear and tear, or damage caused by abuse.
- Clearance bedding sold in 'as is' condition.
- Failure to comply with manufacturer care and maintenance instructions.

IN ORDER FOR THIS GUARANTEE TO BE VALID YOU MUST:

- Be the original purchaser.
- Provide a copy of the original store receipt.
- Not misuse the product (refer to care and maintenance and safety sections).

PRODUCT GUARANTEE

PLEASE TAKE CARE OF YOUR ERGOADJUST DELUX BED BY TAKING THE FOLLOWING PRECAUTIONS:

- Legs and castors should be checked for tightness regularly.
- When transporting - do not over tighten when restraining.
- Maximum recommended weight limits; on a High Low Lift bed 200kg.
- Always lock castors other than when moving or relocating the bed.
- Do not jump on or abuse the bed.
- Do not constantly push the buttons on the controller when not for intended use.
- Use the controller only for adjusting the bed frame for intended use - not for entertainment.
- Lubricate moving parts using a non-corrosive lubricant every 6 months or as required. Avante recommends the use of INOX.
- Beds should be positioned with clearance from walls and other objects so as not to cause damage.

We will not cover damage caused by neglect, mistreatment or general wear and tear of the bed. Regular maintenance is required.

MAKING A CLAIM

To make a claim under this Guarantee please contact your bedding retailer who will initiate the claim process with us. If this cannot be done, please contact Avante Australia Pty Ltd directly via the details on the back of this manual.

Avante Australia will require photos of the Mattress and Base support system to be sent at the time of a claim being made. For details of how to take acceptable photos please contact Avante Australia (contact details on back page).

NOTES

NOTES



Avante Australia Pty Ltd
Unit 2, 29-71 Saltwater Circuit
Narangba, QLD 4504
Phone: 07 3284 3133
Fax: 07 3284 3199
www.avante.com.au