

Aluminium Shower Stool with Padded Seat

SET - UP AND OPERATION

The Aluminium Shower Stool with Padded Seat is manufactured from lightweight aluminium for increased resistance to corrosion with a waterproof padded seat for optimum comfort.

Stool comes fully assembled.



- ⚠ Please read these instructions before use
- ⚠ This stool is designed as a temporary seating aid only; do not use as a step
- ⚠ This product is suitable for use in showers, bathrooms or wet areas; it is not suitable for use in a swimming pool

Adjusting the height

The stool is height adjustable to match individual requirements or uneven surface.

A series of holes along each leg allow seat height to be adjusted from 475 to 630 mm.

⚠ Ideal seat height

Ideal height is when the user's feet rest flat on the floor, with thighs parallel to the ground or slightly angled with hips just above knee level. Knees should never project above the upper leg.



To adjust height, push the height adjustment button on the leg to release the locating button and adjust to the desired height.

Ensure the button is in the correct position and locked in place when the final height is set.

Repeat for all four legs, adjusting height evenly or individually to match an uneven floor surface.

Before use, ensure the stool is stable and resting firmly on the floor surface.



Regular maintenance

The leg clips should be regularly inspected to ensure they are fully inserted and firmly attached. Do not use the stool without all four leg locating buttons inserted, or if the seat is cracked or deformed.

Regularly check the condition of the rubber tips and replace immediately if you see evidence of rips, tears, cracks or other general wear.

Care & cleaning

To clean the seat after each use, wipe with a mild soap and warm water solution and rinse well. Dry with a clean cloth.

For heavier soiling, wipe clean with a diluted bleach solution and rinse well before drying. Do not use water temperatures in excess of 80°C.

It is recommended the frame be cleaned after heavy use or at least once a week. To clean the frame, wipe clean with a soft clean cleanser and warm water solution and rinse well.

Avoid harsh, abrasive cleaning agents or utensils (such as a wire brush) as these can lead to deterioration of the frame and seat.

Aluminium Shower Stool with Padded Seat

Specifications

PRODUCT CODE	BL1171
MAXIMUM USER WEIGHT	120 kg
PRODUCT WEIGHT	2.55 kg
OVERALL WIDTH	505 mm
OVERALL HEIGHT	660 > 760 mm
SEAT HEIGHT	475 > 630 mm
WIDTH BETWEEN ARMRESTS	460 mm
SEAT WIDTH	370 mm
SEAT DEPTH	295 mm
WARRANTY	1 year
ARTG	176831
CONFORMS TO	AS/NZS 3973:2009
MADE IN CHINA	

Limited Warranty

This warranty is provided by **Novis Healthcare** (ABN 45 102 734 491) of Unit 12, 12 Mars Road, Lane Cove West NSW 2066

Novis Healthcare (Novis) products are manufactured to the highest quality standards and are thoroughly tested and inspected before leaving our factory. In addition to any statutory rights and remedies you may have, Novis warrants all of its products sold directly or via an Authorised Novis Australia Dealer against defective workmanship and faulty materials from the date of purchase by the end user for a period of twelve months unless otherwise specified for that product and its components.

Warranty Claims

To claim under this warranty, please contact Novis Healthcare and have your receipt or proof of purchase available. Novis Healthcare may need to assess the defect before determining any claim, and additional information may be requested to process your claim. Claims without proof of purchase may not be able to be processed.

Novis Healthcare may at its option inspect the goods on site or require them to be returned to its premises or one of its Authorised Service Agents in person or freight prepaid by you.

Novis will undertake at its option, to repair or replace, free of charge, each product or part thereof on the condition that:

- The product found on examination, to be suffering from a manufacturing defect;
- The product or relevant part has been serviced regularly by Novis or one of its Authorised Service Agents and has not been subjected to misuse, neglect or been involved in an accident;
- The repairs are not required as part of normal wear and tear.

At our option

- Goods repaired may be replaced by refurbished good of the same type rather than being repaired.
- Refurbished parts may be used to repair goods.

Novis Healthcare will not be held responsible for any repair other than those carried out by it or one of its Authorised Service Agents.

Warranty repairs do not extend the length of the warranty period.

Limited Liabilities

Our liability under this manufacturer's warranty is subject to us being satisfied that a defect was caused by faulty parts, manufacture or workmanship, and was not caused or substantially contributed to by other factors or circumstances beyond our control, including (but not limited to) defective installation, maintenance or repair, product modification or alteration, any neglect, misuse, or excessive use, normal wear and tear or failure to follow manufacturer's instructions.

Important notice for Australian consumers:

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure. To obtain compensation, you will need to provide documentary evidence of the loss or damage suffered and documentary evidence that such loss or damage was a reasonable foreseeable consequence of a failure Novis Healthcare to comply with a consumer guarantee under the Australian Consumer Law. Subject to the provisions of the Australian Consumer Law, Novis Healthcare excludes, to the fullest extent permitted by law, all liability in respect of loss of profit or other economic loss, direct to indirect or consequential, special, general or other damages or other expenses or costs which may include negligence.