



CONTROL PANEL

- Power / Mute**
- Adjust** **Comfort Level Setting**
- Pressure Monitoring** **Pressure Monitoring Indicator LED**
- Alert** **Low Pressure Indicator LED**

OPERATING INSTRUCTION

- Power**

 - Plug the power cord into a wall socket
 - Press to turn on the power unit, and then the green LED will light up

Mute

 - Press to turn off the audible indicator

- Comfort Level Setting**
- Press to adjust the setting from -2 to 2 * The default comfort setting is "auto-set"

- Pressure Monitoring**

Pressure Monitoring Indicator LED

 - The power unit monitors the mattress' pressure 24 hours
 - The yellow LED indicates this function is in operation

Alert

Low Pressure Indicator LED

 - The yellow LED will start flashing with audible indicator when the mattress has insufficient pressure

PROBLEM	CONTROL PROCEDURE	POSSIBLE SOLUTION
Power unit is working, but the mattress is not inflating	Verify the air cells and the compressed air hoses are not damaged	Contact the authorized distributor for technical service
	Confirm that the air cells are connected to the compressed air hoses	Make sure the air cells are firmly connected
	Confirm that there are no pressing points or bending in the compressed air hoses	If necessary, adjust the position of the compressed air hoses
	Confirm that the compressed air hoses are properly connected to the power unit	Make sure the compressed air hoses are firmly connected
No alternating pressure regulation	Power unit failure	Contact the authorized distributor for technical service
	Confirm that there are no pressing points or bending in the compressed air hoses	If necessary, adjust the position of the compressed air hoses
No indications on the control panel, the power unit does not work	Verify that the power cord set plugs into the proper socket	Insert the power cord set into an appropriate socket and turn the power on
	Verify that the power cord set is not defective	Contact the authorized distributor for technical service
Low pressure indicator	Verify the air filter is not obstructed	Clean the air filter and ensure it is not blocked
	Check for leaks in the compressed air system	Contact the authorized distributor for technical service
	Verify that the air pump is not defective	Contact the authorized distributor for technical service