


## BetterLiving Community Bed – Self Help Pole

### SET - UP A N D O P E R A T I O N

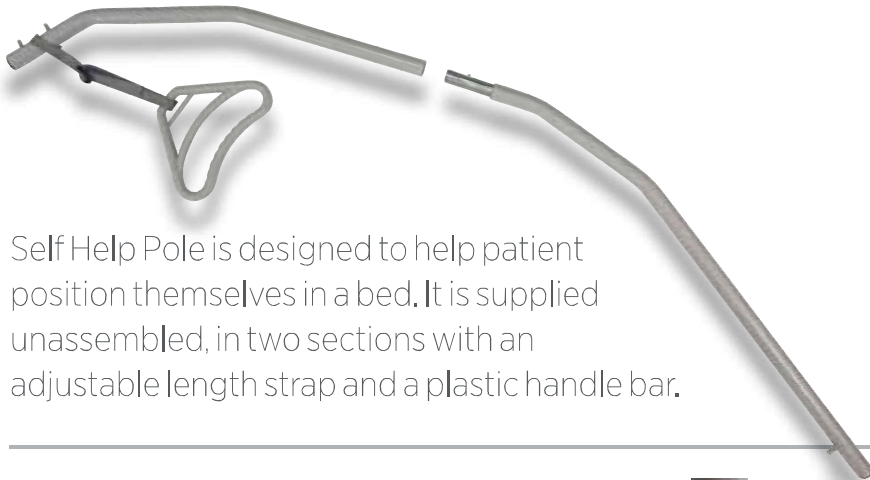

Self Help Pole is designed to help patient position themselves in a bed. It is supplied unassembled, in two sections with an adjustable length strap and a plastic handle bar.

- ⚠ Please read these instructions before use.
- ⚠ Safe Working Load of 75 kg
- ⚠ This accessory is designed for use with the BetterLiving Community Bed only.

Only approved bed accessories listed in the BetterLiving Community Bed Parts List should be used with the bed unless otherwise specified by Novis Healthcare.

#### Assemble Product

- 1 To assemble, attach two sections of the pole by inserting open ends of each section together.
- 2 Align the locking button with the corresponding hole and push in the locking button in as two ends are pushed together
- 3 An audible click can be heard and the locking button head is visible through the hole.

To disassemble, depress the locking button while pulling two sections of the pole away from each other.



#### Regular maintenance

Visual inspection for obvious wear and tear of bar and components.

Replace immediately if you see evidence of tears, cracks or other general wear.

#### Care & cleaning

Hand clean using hand spray and wipe methods only. Novis recommends use of officially recognized disinfectants and mild cleaning solutions.

#### Attach to Bed

Remove or move mattress to locate Self Help Pole bracket

- 1 To attach the pole to the bed, align the notch at base section of the pole to groove in the black plastic sleeve at head end of the bed.
  - 2 With the notch and groove aligned, insert the pole with a direct downward vertical force. Check the notch is in contact with the bottom of the groove.
- ⚠ Note, there are two notches on bed frame, depending on where the pole needs to be positioned for patient usage. Once in a notch, pole will not 'rotate'.

Strap may be adjusted to suit patient comfort



#### Warranty

Novis Healthcare Pty Ltd warrants this product for a period of 12 months. Please refer to our Warranty Statement for full details.

Please contact your authorised dealer or the manufacturer for a list of available spare parts.

#### Specifications

CODE	BE7003
MAXIMUM USER WEIGHT	75 kg
PRODUCT WEIGHT	4.5 kg
HEIGHT	1400 mm
CONFORMS TO	ARTG176883
MADE IN VIETNAM	

#### To Remove

To remove Self Help Pole, pull vertically upwards out of the the black plastic sleeve in the bed frame.

- ⚠ Do not pull at an angle or rotate the pole.

