

Ellipse SuperLite Carbon Fibre Rollator

Product Code: 9346376000628 (Large, Black)

Features:

- ▶ Aust. STD Tested 492484
- ▶ Light Weight Carbon Fibre
- ▶ Added Cane Holder
- ▶ Easily Installable Nylon Bag
- ▶ Adjust Brake Tool-Free
- ▶ Adjust Height Tool-Free














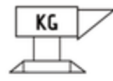
Accessories:

- ▶ Cup Holder- 9346376000574



- ▶ LED Light- LED-C-FX



Length (cm)	 76	Folded Width (cm)	 26	Front Castor Diameter (cm)	 24.2/9.5"
Width (cm)	 60	Seat Depth (cm)	 22	Rear Castor Diameter (cm)	 20.3/8"
Max Height (cm)	 105	Seat Width (cm)	 45.4	Max Load (kg)	 120
Minimum Height (cm)	 92	Seat Height from the Ground (cm)	 60	Net Weight (kg)	 6.69

User Manual

Ellipse Super Lite Rollator



Carbon Fibre Rollator

List of Parts




- | | |
|-----------------------------------|------------------------|
| 1. Handgrip | 7. Seat |
| 2. Adjustable tubing hand grip | 8. Bag |
| 3. Rear reflector handgrip | 9. Front wheel |
| 4. Brake lever | 10. Brake adjust screw |
| 5. Interlock button for handgrips | 11. Rear wheel |
| 6. Back Strap | 12. Safety bow |

This pack includes


<p>1 Ellipse Super Lite Rollator</p> 	<p>2 Bag</p> 	<p>3 Back Strap</p> 	<p>4 User Manual</p> 
---	---	---	---

UNFOLDING ELLIPSE SUPER LITE

Position Ellipse on its safety bow and lift it

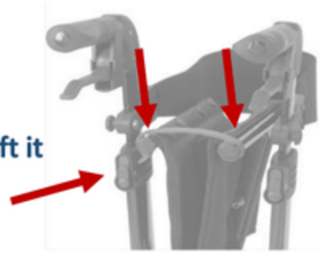

Push downwards on the seat rails to engage the locking mechanism.

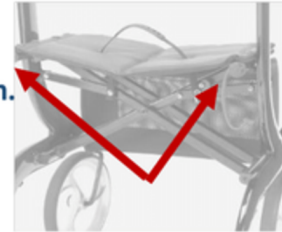




HEIGHT ADJUSTMENT

Press the red release button located below the handgrips as shown and at the same time either push the handle down (lowering handle height) or pull the handle up (raising handle height), releasing the push button will automatically lock the handle at desired height.



ATTACHING THE CARRY BAG

Fold Ellipse Super Lite, Remove the stopper from the tube on either side of the seat, push the sliding block from bag into the guiding sleeve and put the stopper back in place

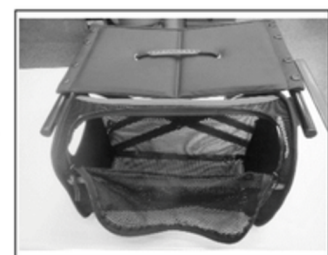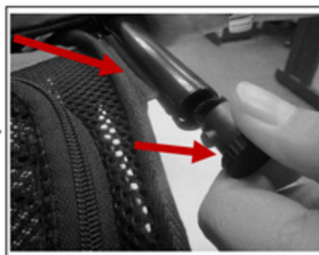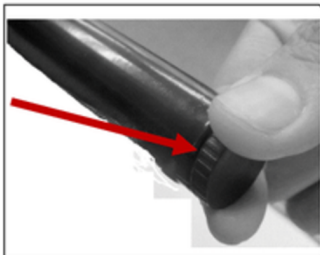

BRAKING / LOCKING

Pull both brake levers upwards to apply the brakes while the Ellipse Super Lite is moving. Always brake slowly and steadily





Press the end of the brake lever as shown to apply the hand brake. Pull up the end of the lever to release the hand brake.



The effect of the brakes can be affected by abrasion of the wheels. Test the brakes regularly.

ADJUSTMENT OF BRAKES

To adjust the braking strength, adjust the brake by pressing the screw housing downwards and then left or right

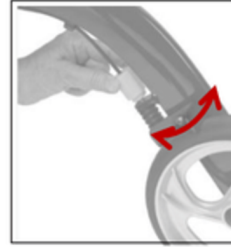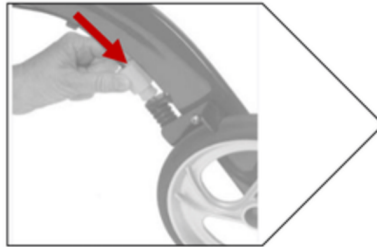

SITTING

Position the Ellipse Super Lite on an even and solid surface. Lock the brakes (parking position). The Ellipse Super Lite must stand and must not slip. Sit straight up and do not lean backwards



FOLDING/ TRANSPORT/ STORAGE

To fold, pull seat upwards by the handle of the seat plate



Once the Ellipse Super Lite is folded, use the red safety clip to secure the seat. This will stop the walker from opening whilst lifting in or out of a vehicle. See below:



SAFETY INSTRUCTIONS

IMPORTANT → Do not propel yourself or have anyone push you whilst you are seated on the Rollator. This is a walking aid only and is not to be used as a transportation device.

- Use only as walking aid
- Use only on even and solid surfaces
- Maximum user weight: 120 Kg
- Regularly check the screws and fasteners on the Ellipse Super Lite and retighten if necessary.
- Use bag only for transport of objects and do not overload. Maximum load: 5 Kg
- Do not expose the product to temperatures above 40 °C for an extended period of time, as this can cause the Ellipse Super Lite parts to overheat and burn the user
- Please make sure the Rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
- Ensure that both knobs are fully tightened and are at equal heights. Test Rollator for stability before applying full weight to hand grips.
- Test the brake lever before operating the Rollator. When wheels are locked, lock bar on rear wheels should prevent wheels from turning. When locks are released, the wheels should turn freely.
- Always lock the brake lever before sitting and do not use seat when unit is on an inclined or uneven ground.
- Replace castors when tyres are significantly worn or damaged.

CLEANING AND MAINTENANCE

Clean regularly by hand with a damp cloth. Do not use aggressive detergents.

SPECIFICATIONS

	Small	Medium	Large
Overall Length (mm)	660	680	706
Overall width (mm)	600	600	600
Handle (highest) (mm)	850	960	1050
Handle (lowest) (mm)	760	835	920
Seat depth (mm)	220	220	220
Seat Width (mm)	450	454	454
Seat height (mm)	480	520	600
Product nett weight (approx.) (kg)	6.26	6.63	6.69
Weight Limit (kg)	120	120	120
Wheel diameter Rear (mm)	203	203	203
Wheel diameter Front (mm)	203	242	242



Level 1, 149 Princes Highway
Unanderra NSW 2526
Australia
Tel: +61 2 4272 2688. Fax: +61 2 4272 2699
www.peak-care.com.au

Distributor:



- Page intentionally left blank